


**Low Incidence Needs Team**  
**Hearing Impairment – Care and Maintenance of**  
**Audiological Equipment**

# Hearing Aid Check

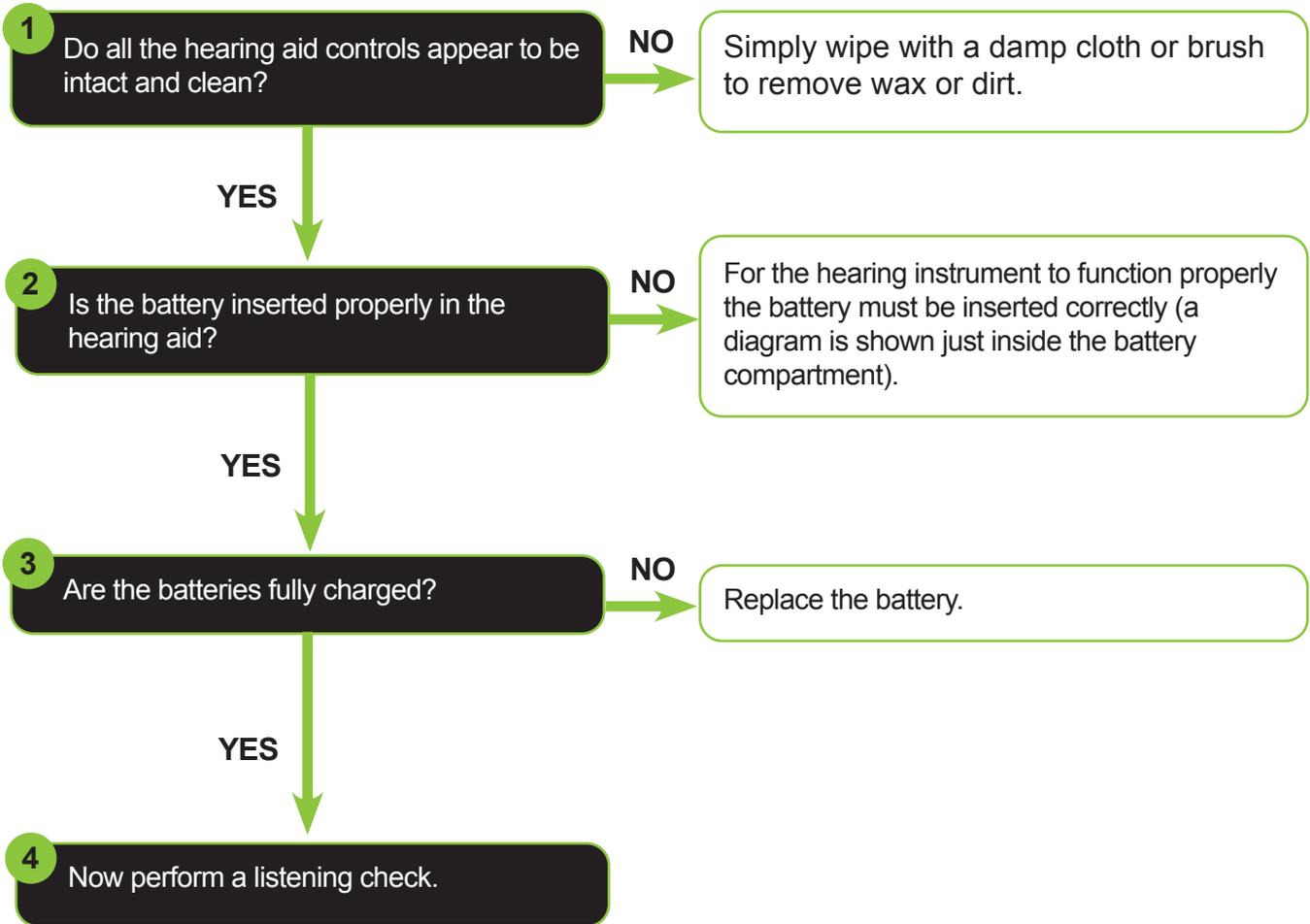


(Please see next page - Daily Listening Checks)

**Audiological equipment should be checked daily.**

# Daily Listening Check

To protect your own hearing lower the volume by using the attenuator on the stetoclip prior to your listening check.

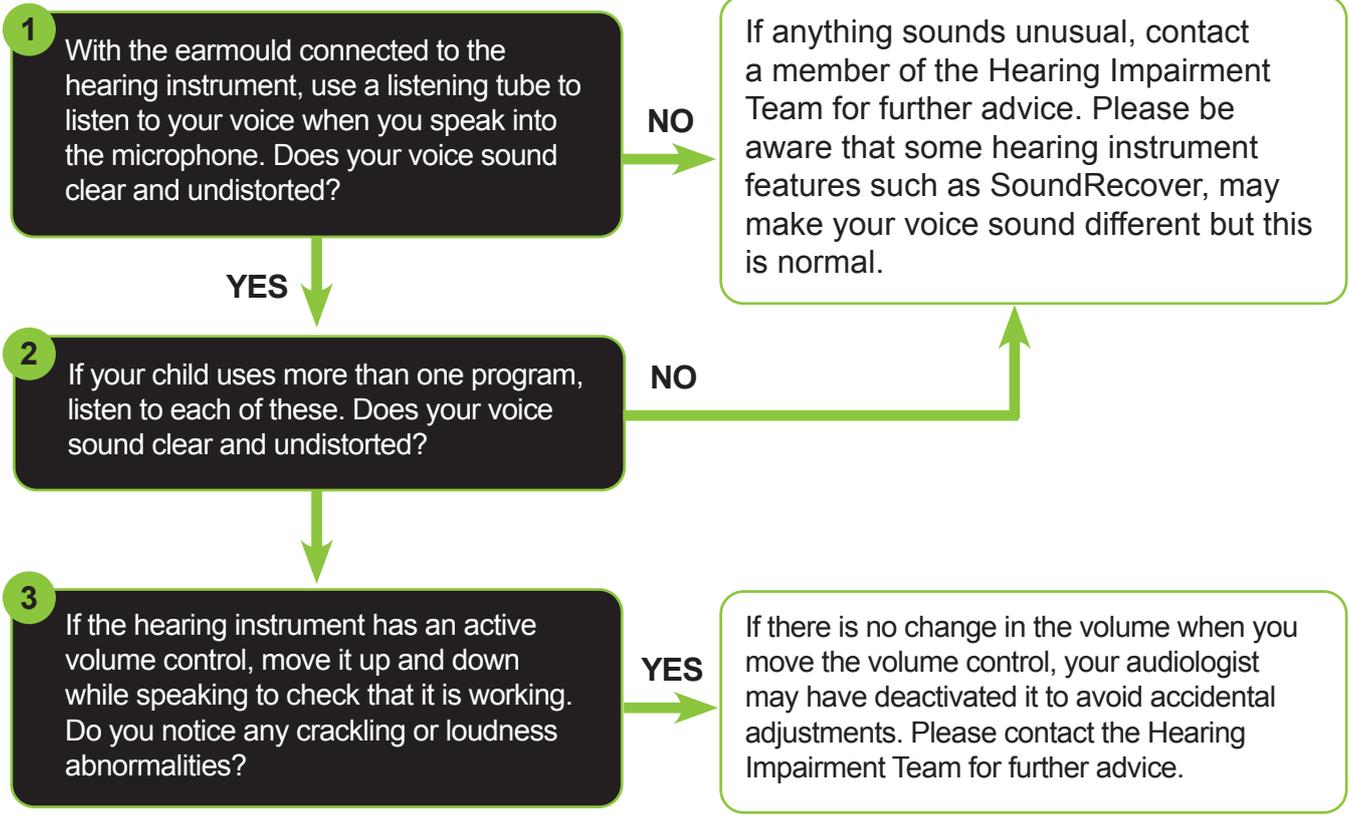

**Congratulations - you have completed your daily listening check!**

## Listening Check Troubleshooting

When listening to through the aid, via the earmoulds, if the hearing aid is not working or sounds distorted please use the following to specifically diagnose the problem:

Remove the mould and tubing and listen through the hook and aid. If it sounds clear, clean and blow air through the tubing and mould to clear any obstruction.

Most problems can be rectified by removing wax, blowing out condensation or re-tubing the earmoulds but if it is a problem with the aid itself it will have to be taken back to the hospital, by parents, to be checked.

**Audiological equipment should be checked daily.**